

Schine Pill Popper

Schine Pill Popper

- Makes it easy to pop tablets out of blister packaging.
- Ergonomic design.
- The rotatable section adapts to different sizes of tablet.
- Promotes good hygiene.
- Dishwashable. Withstands alcohol-based cleaning.

Many people find it difficult to pop tablets out of blister packages, and loading pill boxes with the correct dosage can be tricky.

The Schine Pill Popper is a device which makes it easier to handle pills. Simply pop the tablet out of the blister package and make sure it enters the intended chamber via the transparent, detachable tube.


A hygienic solution which also saves time. The Pill Popper meets the ergonomic standards relating to hand-held devices when it comes to the thickness of the handle, length and need of manual input.

Approved by the Swedish Rheumatism Association as a product which is easy to use.

Recommendations for users

- Before using the product for the first time, clean it using warm water and detergent, or dishwasher, at a maximum of 84°C.
- Tablets which break up must be disposed of.
- The Schine Pill Popper can be used for most types of packaging. Where problems arise, for example, if the foil is stubborn, or when handling extremely large or small tablets, we advise users to work manually.
- The supplier shall not be liable for incorrect use of the Schine Pill Popper, or where dosage of medicine is incorrect.

Specifications

Weight:	90 g	
Length:	20 cm	
Width:	2 cm	
Height:	12 cm	
Material:	Recyclable plastic	
More:	Dishwashable at a maximum of 84°C, and withstands alcohol-based cleaning.	

